**(UK) - INSTRUCTIONS FOR USE: OESOPHAGEAL DOPPLER PROBES**

**1. Indications** – The TrueVue™/ODM+™ system's beat-to-beat data on cardiovascular status are used by the managing clinician to evaluate and optimise haemodynamic performance of patients who are; in surgery and are anaesthetised or sedated; in intensive care; in the emergency room; in the obstetrics department; in other wards or departments where haemodynamic measurements are required. Before use read the appropriate Operating Handbook. Use only with the appropriate Monitoring System.

**2. Intended Use:**  
The purpose of the TrueVue/ODM+ system is to measure and calculate the haemodynamic parameters used to assess Preload, Afterload and Contractility of the heart on a beat-to-beat basis in real time. Parameters related to haemodynamics including Stroke Volume, Stroke Distance, Cardiac Output, Peak Velocity, Systemic Vascular Resistance, Mean Arterial Pressure, Pulse Pressure Variation, Stroke Volume Variation are examples of measurements and calculations to be made available to clinicians managing patient haemodynamics.

**3. Clinical benefit**- Proven effective for haemodynamic optimisation to reduce complications and shorten length of hospital stay

**4. Warnings - Do Not Use in the Following Situations - Patient Harm May Result:**

- Do not use after the expiry date printed on the shelf box and unit pack labels.
- Do not use undue force upon insertion. Remove if difficulty arises during insertion and seek advice.
- Verify coagulation status for the possibility of nasal bleeding prior to inserting.
- Use only by trained and qualified personnel, or under such supervision.
- Not recommended for use in patients under going intra-aortic balloon pumping.
- The probe may be unreliable during periods of aortic cross clamping.
- Any serious incident that occurs while using this product shall be reported to Deltex Medical Ltd and your member state competent authority.
- MR unsafe, do not use this equipment in the MRI scan room
- Single patient use only. Risk of cross contamination if probe is used on more than one patient

**5. Cautions:**

Do not use if the pouch has been unintentionally opened, or if the probe or pouch show signs of damage.

The probe shall be disposed of according to local procedures. Exercise caution when handling. The Probe contains an internal spring that will cause the probe to uncoil when released from the pouch.

Open the pouch at the chevron seal end.

Do not use scissors or sharp tools to open the pouch.

Do not use if the pouch has been opened, or if the probe or pouch show signs of damage.

Clinicians are advised to take into consideration the potential of increased risks of pressure on the oropharynx when used in combination with other equipment.

Should the probe be removed and stored for same patient reuse, refer to local guidelines for cleaning and storage.

**6. Contraindications:**

Not for use on patients under 16 years of age.

Do not use with carcinoma of the pharynx, larynx, oesophagus.

Do not use with aneurysms of the thoracic aorta.

Do not use with proximal coarctation of the aorta.

Do not use with tissue necrosis of the oesophagus or nasal passage.

Do not insert nasally where there is nasal injury; where there is facial trauma; nasal polyps or where there may be the risk of brain injury.

Do not use in close proximity to laser surgery.

Not for use in patients with pharyngo-oesophago-gastric pathology and/or severe bleeding diatheses.

**7. General Information:**

The Doppler Probes are supplied STERILE.

Probe output specifications are documented within the TrueVue System/CardioQ-ODM+ Monitor handbooks

**8. Removal of the Probe from the Unit Pack:**

Open the pouch at the chevron seal end and, holding the Probe withdraw it from the pouch. Allow the Probe to uncoil, keeping it free from contact with any object that may compromise its state of cleanliness.

**9. Insertion of the Probe into the Patient:**

Insertion should be carried out by an appropriately trained and qualified medical practitioner or by a person under the supervision of such.

Insertion of the Probe may be either orally or nasally. Note the Warnings above.

The Probe connector should be plugged into a Dopplink (TrueVue System) or a Patient Interface Cable connector (CardioQ-ODM+) and the correct nomogram entered.

Apply a water-based lubricant to the tip of the Probe. This aids insertion and signal acquisition. Insert the Probe with the bevelled edge of the tip facing the back of the throat so that when in the oesophagus, the bevelled edge faces backwards and downwards towards the descending aorta.

Do not use excessive force upon insertion. Remove if difficulty arises and seek advice.

Warning risk of patient harm if forced.

The numbered depth markers on the Probe shaft are for use as a guide for oral and nasal placement. With oral placement, signal acquisition may normally be achieved at a depth of between 35cm (marker 1) and 40cm (marker 2) in an adult patient. With nasal placement, signal acquisition may normally be achieved at a depth of between 40cm (marker 2) and 45cm (marker 3) in an adult patient. The required depth of insertion will depend upon patient height/weight. Patient characteristics will vary between individuals; use both direct visual and audible focusing to achieve the optimum ultrasound signal.

Refer now to the appropriate Operating Handbook for positioning the Probe and operating the monitor. The total probe usage time remaining is displayed when the Probe is connected.

**10. Removal of the Probe from the Patient:**

When temporarily withdrawing the probe from a patient (e.g., when resting or moving the patient), the probe can be disconnected from the Dopplink (TrueVue System) or Patient Interface Cable (CardioQ-ODM+). If the period of removal is transitory, the probe should be cleaned using a sterile wipe before re-insertion.

**11. Probe Disposal:**

Used probes should be disposed of in accordance with the appropriate Health Authority, or similar controlling bodies, guidelines for clinical waste.

**12. Storage:**

Store in an enclosed box between -20°C and 60°C.

Do not expose to direct Ultra Violet (UV) light, strong smells or dampness/wet.

Manufactured By Deltex Medical Ltd, Terminus Road, Chichester, PO19 8TX, UK

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Proizvodi Deltex Medical Ltd, Terminus Road, Chichester, PO19 8TX, UK
Consultancy Services Ltd.
medReg GmbH